

Product information DIAGNOcam



KaVo. Dental Excellence.

DIAGNOcam connection with USB interface for KaVo treatment units Estetica E80, E70, E50 and other manufacturers.

Estetica E 80

The Estetica E 80 has always a USB connection included in the dentist and assistant element. Through the USB interface the DIAGNOcam can be connected to the dentist or assistant element and can be put down either on the free instrument holder or on the DIAGNOcam holder which is included in the package.

Estetica E70

The Estetica E 70 does not have the USB connection as a standard configuration. It has to be ordered specifically.

If the USB connection was not considered in the configuration, a retro fit kit can be installed at the dentist element.

Retro fit kit

Material number: 1.006.9747; KaVo List price: 165,-€

Please keep in mind:

This is only possible after serial number SN 2010000. Units with serial number before 2010000 the retro fit kit cannot be installed.

Estetica E 50

The Estetica E 50 does not have the USB connection as a standard configuration. It has to be ordered specifically.

If the USB connection was not considered in the configuration, a retro fit kit can be installed at the dentist element.

Retro fit kit

Material number: 1.006.9747; KaVo List price: 165,-€

The installation of the retro fit kit has to be completed by a service technician.

Please keep in mind:

Treatment units from the product line „ESTETICA“ which got delivered between March 2009 and August 2010 could have transmission problems. By changing the USB cable (1004.6953, List price 67,40 €) the problems will be solved.

The identification of the cable which could cause transmission problems can be easily tested on the PC.

Connect the loose end of the USB cable from the connection box (see picture) with your PC.

Cable with the indication:

- Vid_05e3 and Pid_0608 or
- Vid_0409 and Pid_0059

do not have to be changed.

All other KaVo treatment units do not have an USB connection.

Other manufacturers:

In generally the DIAGNOcam can be integrated in other treatment units which fulfill the specification of the USB 2.0. The correct function has to be guaranteed by the service technician.

